

## PANEL STRIPS

Blue bars below represent test panels strips built into Identify Diagnostics cup and dip tests. Your test may include one or more panels based on your specific configuration.

## BRAND® / STREET NAMES

Green tabs below are common drug brands or street names. Not all drug brands or street names are included in this list. The list of drug compounds under each blue bar indicate what drugs are detectable by that panel and at what cut-off level in ng/mL.

DRUG COMPOUND	Cut-off in ng/ml	DRUG COMPOUND	Cut-off in ng/ml
<b>AMPHETAMINE 1000 AMP</b>		<b>MARIJUANA 50 THC</b>	
D-Amphetamine <b>ADDERALL®</b>	1,000	11-nor- $\Delta^8$ -THC-9-COOH <b>POT, WEED</b>	50
D,LAmphetamine	2,500	11-nor- $\Delta^9$ -THC-9-COOH <b>POT, WEED</b>	50
L-Amphetamine	50,000	$\Delta^8$ -Tetrahydrocannabinol	8,000
( $\pm$ )3,4-Methylenedioxyamphetamine (MDA)	2,000	$\Delta^9$ -Tetrahydrocannabinol	10,000
Ephedrine	>100,000	Cannabinol	10,000
3,4-Methylenedioxyethylamphetamine (MDEA)	>100,000	Cannabidiol	100,000
<b>BARBITURATES 300 BAR</b>		<b>METHADONE 300 MTD</b>	
Secobarbital <b>SECONAL®</b>	300	Methadone <b>DOLOPHINE®</b> <b>METHADOSE®</b>	300
Phenobarbital <b>LUMINAL®</b>	2,500	Doxylamine <b>DICLECTIN®</b>	50,000
Butalbital <b>FORTABS®</b>	500	<b>METHAMPHETAMINES 1000 MET</b>	
Pentobarbital <b>NEMBUTAL®</b>	1,500	+/-Methamphetamine <b>DESOXYN®</b>	2,000
Amobarbital <b>AMYTAL®</b>	2,500	+Methamphetamine	1,000
Cyclopentobarbital	500	3,4-Methylenedioxyethylamphetamine	35,000
Butethal <b>FIORICET®</b>	800	(+/-)3,4-Methylenedioxymethamphetamine (MDMA)	2,000
Barbital <b>VEVONAL®</b>	300	Ranitidine <b>ZANTAC®</b>	>100,000
Butobarbital <b>BUTISOL®</b>	1,500	3,4-Methylenedioxyamphetamine (MDA)	>100,000
<b>BENZODIAZEPINES 300 BZO</b>		D-Amphetamine	>100,000
Oxazepam <b>SERAX®</b>	300	L-Amphetamine	>100,000
Alprazolam <b>XANAX®</b>	200	Ephedrine	>100,000
a-Hydroxyalprazolam	1,000	<b>METHYLENEDIOXYMETHAMPHETAMINE 500 MDMA</b>	
Bromazepam <b>LEXOTANIL®</b> <b>LECTOPAM®</b>	250	(+/-)3,4-Methylenedioxymethamphetamine (MDMA) <b>ECSTASY</b>	500
Chlordiazepoxide <b>LIBRIUM®</b>	2,500	D-Amphetamine	>100,000
Clobazam <b>ONFI®</b>	100	L-Methamphetamine	100,000
Clonazepam <b>KLONOPIN®</b>	850	3,4-Methylenedioxyethylamphetamine (MDEA)	200
Clorazepate <b>TRANXENE®</b>	250	3,4-Methylenedioxyamphetamine (MDA)	2,000
Delorazepam <b>DADUMIR®</b>	1,600	<b>OPIATE 300 MOP</b>	
Diazepam <b>VALIUM®</b>	200	Morphine <b>HEROIN METABOLITE</b> <b>ROXANOL®</b>	300
Estazolam <b>PROSOM®</b>	200	Codeine <b>TYLENOL 3®</b>	300
Flunitrazepam	300	Hydrocodone <b>LORATAB®</b> <b>LORCET®</b> <b>VICODIN®</b>	2,000
Lorazepam <b>ATIVAN®</b>	1,000	Hydromorphone <b>DILAUDID®</b>	3,500
Midazolam	1,500	Morphine 3- $\beta$ -D-glucuronide	300
Nitrazepam	100	6-Monoacetylmorphine <b>HEROIN METABOLITE</b>	600
Nordiazepam	400	Normorphine	100,000
Temazepam <b>RESTORIL®</b>	150	Oxycodone <b>OXYCONTIN®</b>	10,000
Triazolam <b>HALCION®</b>	500	Oxymorphone <b>OPANA®</b>	50,000
<b>BUPRENORPHINE 10 BUP</b>		Thebaine	7,000
Buprenorphine <b>SUBOXONE®</b> <b>SUBUTEX®</b>	10	<b>OXYCODONE 100 OXY</b>	
Norbuprenorphine	15	Oxycodone <b>OXYCONTIN®</b> <b>PERCOCET®</b>	100
Buprenorphine-3-D-glucuronide	12.5	Morpine <b>HEROIN METABOLITE</b> <b>ROXANOL®</b>	50,000
Norpuprenorphine-3-D-glucuronide	175	Codeine <b>TYLENOL 3®</b>	25,000
Morphine-3-D-glucuronide	100,000	Morphine 3- $\beta$ -D-glucuronide	50,000
Morpine <b>HEROIN METABOLITE</b>	>100,000	Hydrocodone <b>LORATAB®</b> <b>LORCET®</b> <b>VICODIN®</b>	1,600
Oxymorphone <b>OPANA®</b>	>100,000	Hydromorphone <b>DILAUDID®</b>	15,000
Hydromorphone <b>DILAUDID®</b> <b>OPIDOL®</b>	>100,000	Normorphine	100,000
<b>COCAINE 300 COC</b>		Oxymorphone <b>OPANA®</b>	1,500
Cocaine <b>COKE, CRACK</b>	>100,000	<b>PHENCYCLIDINE 25 PCP</b>	
Benzoylcegonine <b>COCAINE METABOLITE</b>	300	Phencyclidine <b>ANGEL DUST</b>	25
Ecgonine HCl	35,000	<b>SYNTHETIC MARIJUANA 25 K2</b>	
<b>COTININE 200 COT</b>		Synthetic Marijuana <b>SPICE</b>	25
Cotinine <b>NICOTINE</b>	200	<b>TRAMADOL 100 TRA</b>	
<b>EDDP 300 EDDP</b>		Tramadol <b>CONZIP®</b> <b>ULTRACET®</b> <b>ULTRAM®</b>	100
EDDP Methadone Metabolite	300	<b>TRICYCLIC ANTIDEPRESSANTS 1000 TCA</b>	
<b>ETHYL GLUCURONIDE 500 ETG</b>		Notriptiline <b>PAMELOR®</b>	1,000
Ethyl Glucuronide <b>ALCOHOL</b>	500	Trimipramine <b>SURMONTIL®</b>	4,500
<b>FENTANYL 25 FEN</b>		Amitriptyline <b>ELAVIL®</b>	1,000
Fentanyl <b>DURAGESIC®</b> <b>ABSTRAL®</b> <b>LONSY®</b>	25	Promazine <b>SPARINE®</b>	3,000
<b>GABAPENTIN 2000 GAB</b>		Desipramine <b>NORPRAMIN®</b>	1,000
Gabapentin <b>GRALISE®</b> <b>NEURONTIN®</b>	2,000	Imipramine <b>TOFRANIL®</b>	1,000
<b>KRATOM 100 KRA</b>		Clomipramine <b>ANAFRANIL®</b>	7,500
Kratom	100	Doxepin <b>SILENOR®</b> <b>ZONALON®</b>	3,000
		Maprotiline <b>LUDIOMIL®</b>	50,000